

Golamir



2ACT

**Reduces pain
and protects
the throat**

100%
natural

gluten
free

ALCOHOL-FREE SPRAY



ADULTS AND CHILDREN
FROM 1 YEAR OF AGE


MEDICAL DEVICE Class IIa

Golamir 2Act alcohol-free spray is an innovative Medical Device that soothes pain and irritation through an **indirect anti-inflammatory action** that **protects** the oropharyngeal mucosa. It is indicated for sore throats to treat irritation and inflammation caused by viruses, bacteria, allergens or contact with external irritants (smoke, smog and dust).


The product can also be used for colds when post-nasal drip may cause throat irritation.

Thanks to its natural active complex, **ActiFilm DOL**, **Golamir 2Act alcohol-free spray** acts on pain, burning and difficulty swallowing by protecting the mucosa from the factors that trigger irritation.

ActiFilm DOL, consisting of tannins, resins, polysaccharides and flavonoids, forms an **active, dual-action film that acts as a barrier** and:

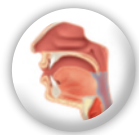

Protects the mucosal surface, reducing hypersensitivity

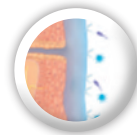

Normalises the mucosa through **indirect anti-inflammatory activity**

The tannin and resinoid fractions in **Golamir 2Act alcohol-free spray** also create an unfavourable environment for bacterial growth.

The product is 100% natural, gluten-free and has a pleasant taste of Forest fruits and Blackberries.

INDICATIONS

Golamir 2Act alcohol-free spray is indicated for sore throats in the treatment of irritation and inflammation of the oropharyngeal cavity, which may be caused by viruses, bacteria, allergens or contact with external irritants (smoke, smog and dust).

The product can also be used for colds when post-nasal drip may cause throat irritation.

The product can be used from 1 year of age.

Given its mechanism of action, **Golamir 2Act alcohol-free spray** can also be used concomitantly with medication prescribed by a doctor.

COMPOSITION

Functional substances: ActiFilm DOL, a natural active complex composed of: tannin and flavonoid fraction from Agrimony, polysaccharide fraction from Aloe vera leaf gel and resinoid fraction from Myrrh.

Also contains: Vegetable glycerine, Water, Apple juice, Lemon juice, Forest fruits and Blackberry natural flavourings and Xanthan gum.

INSTRUCTIONS FOR USE

It is recommended to administer (in the oropharynx):

- **4 sprays up to 4 times a day for children (from 1 to 6 years of age);**
- **4 sprays up to 8 times a day for adults and children (over 6 years of age).**

In younger children, the product can be sprayed on the inside of the cheeks so that it is gradually absorbed. Shake the bottle before use.

Inside the carton box there is a nozzle in a sealed sachet. On first use, remove the aluminium cap from the bottle and replace it with the nozzle.

WARNINGS

Do not use in case of individual hypersensitivity or allergy to one or more components. If symptoms persist, seek medical advice.

Store at room temperature, away from heat sources and protected from light.

Keep out of reach of children.

The expiry date refers to the intact product, stored correctly.

PACKAGE

30 ml glass bottle with spray nozzle.

EXPIRY DATE

See the expiry date stated on the pack. Do not use after the expiry date indicated.

We recommend using the product within 3 months of opening.

LAST REVISION DATE: 12/2017



Aboca S.p.A. Società Agricola

Località Aboca, 20 - 52037 Sansepolcro (AR) - Italia - www.aboca.com

Company operating under certifications ISO 9001 and ISO 13485

for quality management system and ISO 14001 for environmental management system.