

# Fitonasal Pediatric

MT

Decongests and protects the mucosa

NEBULISER SPRAY WITH A NON-PHARMACOLOGICAL  
DECONGESTANT ACTION

Ideal for:

- Nasal congestion
- Coughs resulting from a cold

BALANCED AND STANDARDISED HYPERTONIC SOLUTION



With **Tannisal**  
Natural molecular complex  
of Tannins and Rock Salt



MEDICAL DEVICE Class IIa

**Fitonasal Pediatric** is an **innovative decongestant** specially formulated to unblock the nose in cases of congestion and irritation of the nasal cavities caused by flu-like illnesses, the common cold, allergic rhinitis and both acute and chronic sinusitis.

**Fitonasal Pediatric** works thanks to **TANNISAL**, a natural molecular complex of tannins and rock salt, which interacts with the nasal mucosa to provide:

## 1. A decongestant action with a dual non-pharmacological mechanism

- **Precipitation of bradykinin:** the tannins contained in the product are able to bind to this inflammatory mediator, thus reducing nasal congestion;
- **Balanced and standardised osmolality** (600 mOsm/kg), which helps water to be drawn back from the congested tissues, without irritating or drying the mucosa.

## 2. Protective effect

- that normalises the nasal mucosa thanks to the adhesive properties of the active substances and their effective antioxidant action.

Moreover, thanks to the product's continuous nebulised spray, it performs a **deep cleansing of the nasal cavities, removing excess mucus and the pathogens, irritants and allergens** trapped within it (*viruses, bacteria, smoke, smog, pollen, irritants, etc.*).

**This action is particularly helpful for infants who are unable to blow their noses and children who do so inadequately.**

Thanks to these characteristics, **Fitonasal Pediatric** is especially indicated in cases of **cough caused by post-nasal drip**. In fact, the build-up of mucus in the nasal cavities and the presence of microorganisms and allergenic substances can, as a result of post-nasal drip, cause cough or other problems such as **sore throat** and **otitis media**.

**Fitonasal Pediatric** does not contain vasoconstrictive decongestants and, thanks to its innovative non-pharmacological mechanism of action, does not create dependency, does not dry out the nasal mucosa, is non-irritating and can be used frequently, even over prolonged periods.

When used in infants and children, the product aids breathing and makes it easier to sleep at night.

## PACKAGE

**Fitonasal Pediatric** comes in a container featuring a spray dispenser with "**bag-on-valve**" (BOV) technology, which is an intrinsic element of the product's effectiveness, offering a variety of benefits:

- **Easy to use** - the BOV technology enables a **continuous-flow spray in all positions**, which is particularly helpful for very young children;
- **Ergonomic** - the spray nozzle has been designed to fit the noses of even very small children;
- **Quality** - the product is contained inside a bag that is isolated from the external environment. This allows product out, but does not allow air, liquids or other residues back in;
- **Environmental protection** - Contains nitrogen, a propellant which respects the ozone layer.

## INDICATIONS

**Fitonasal Pediatric** is indicated in all cases of congestion and irritation of the nasal cavities caused by flu-like illnesses, colds, allergic rhinitis and both acute and chronic sinusitis.

It can also be used as an adjuvant treatment in cases of cough, sore throat caused by post-nasal drip, and otitis media resulting from nasal congestion. Ideal for frequent use.

The product is indicated for children from 6 months and up.

## INSTRUCTIONS FOR USE

Use at least **twice a day**, morning and evening, and any time you feel it is necessary. For an effective decongestant and protective action on the mucosa, use **Fitonasal Pediatric** as follows:

### From 6 months to 1 year of age



1. Lay your child down on one side, put the nozzle into the nostril that is now in the higher position and spray the product.



2. Turn your child onto their other side and repeat in the other nostril.

### Over 1 year of age



1. Put the nozzle into the nostril and apply gentle pressure for a few seconds.



2. Clean the nose or, if possible, blow it before spraying the product into the other nostril.

After each use, clean the nozzle with a clean, damp cloth and dry it thoroughly. **Features a comfort nozzle - suitable even for very young children's noses.**

## COMPOSITION

**Functional substances: TANNISAL**, a standardised, natural, molecular complex composed of a tannic fraction from Witch Hazel and Rock Salt (salt of mined extraction, not chemically treated).

**Also contains:** Water, Vegetable glycerin, Lemon juice, Nahcolite (Natural sodium bicarbonate).

**The product is 100% natural and does not contain preservatives.**

## WARNINGS

Do not use in case of individual hypersensitivity or allergy to one or more components. If symptoms persist, seek medical advice. Use only nasally. Avoid contact with the eyes. If contact occurs, rinse with water.

**Warning** - Pressurised container: may burst if heated. Do not pierce or burn, even after use. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Protect from sunlight. **Do not expose to temperatures exceeding 50°C/122°F.** Keep out of reach of children.

Store in a cool, dry place, away from heat sources and protected from light. The expiry date refers to the intact product, stored correctly.

## EXPIRY DATE

See the expiry date stated on the pack. Do not use after the expiry date indicated.

The expiry date indicated on the pack remains unchanged even after first use.

**PACK FOR SALE:** 125 ml nebuliser spray

**LAST REVISION DATE:** 12/2019



**Aboca S.p.A. Società Agricola** - Loc. Aboca, 20 - 52037 Sansepolcro (AR) - Italia  
**www.aboca.com**

Company operating under certifications ISO 9001 and ISO 13485 for quality management system and ISO 14001 for environmental management system.

**Distributor:** Pharma.MT Ltd, 103 Stuart Street, Gzira, GZR 1054, Malta