Fitonasal Spray

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Corporation

Fitonasal Spray

- Decongests and fluidifies
- Protects and promotes the regeneration of the mucosa

REFRESHING EFFECT

Preservative-free

Propellant-free

Also suitable for prolonged use

Non-pharmacological decongestant action

MEDICAL DEVICE Class IIa

750
mOsm/kg

With a natural molecular complex
of Tannins, Flavonoids and Rock salt

CONCENTRATED SPRAY

EN Fitonasal Concentrated Spray is a **protective decongestant** for use in all cases of **colds, allergies** and for **congestion and irritation of the nasal cavities** in general, **related to both acute and chronic conditions**. Fitonasal Concentrated Spray is a medical device which owes its effect to the combination of the components in the formula. Of particular importance among these is the natural molecular complex composed of Tannins, Flavonoids and Rock Salt, which together with vegetable glycerine gives the product the following characteristics:

1. a non-pharmacological decongestant action, due to its:

- **local astringent action** produced as a result of the product's non-specific interaction with proteins, including the inflammatory protein bradykinin found in the nasal fluids, enabling its removal.
- **balanced and standardised osmolality (750 mOsm/Kg)**, which helps fluids to be drawn back from the congested mucosa, fluidifying the mucus and facilitating its removal, including through the physical action of rinsing the nasal cavities, without causing irritation or dryness.

2. a protective effect, thanks to the mucoadhesive and antioxidant action which helps to maintain the integrity of the epithelial barrier and regenerate the mucosa.

The natural flavours of Eucalyptus and Mint give a pleasant sensation of freshness.


Fitonasal Concentrated Spray does not contain vasoconstrictive decongestants and, thanks to its innovative non-pharmacological mechanism of action, does not create dependency, does not dry the mucosa, is non-irritating and can be used frequently, even over prolonged periods. Product preservation is guaranteed by the production process that uses a sterilising filtration system and by packaging in a microbiologically controlled environment, without the use of preservatives.

PACKAGE: The nasal solution Fitonasal Concentrated Spray is contained in a glass bottle fitted with a nozzle with **APF (Advanced Preservative Free) technology**, a multidose system with an integrated filter membrane ensuring:

- **a pre-dosed nebulised spray** – the pump-spray nozzle releases a specific amount of solution with each spray, ensuring equal doses
- **quality without preservatives** – the filter membrane preserves the microbiological quality of the product at each application, filtering the incoming air. This feature allows the product to be used up to the expiry date without the need to use artificial preservatives
- **environmental protection** – this product does not contain any propellants, thus respecting the environment

INDICATIONS: Fitonasal Concentrated Spray is indicated in all cases of **congestion and irritation of the nasal cavities** caused by colds, even when associated with flu-like syndromes and allergies (seasonal and perennial). It is also of use when these conditions occur in cases of viral and bacterial rhinosinusitis and mechanical-obstructive related rhinitis (deviations of the nasal septum). Thanks to its characteristics, the product is suitable for both frequent and prolonged use. The product can be used from 6 years of age.

DIRECTIONS

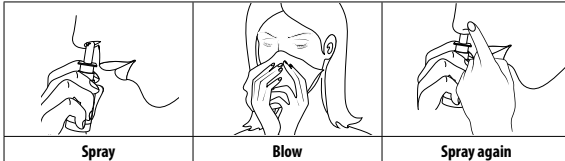Dosage for adults and adolescents over 12 years: Spray twice in each nostril, blow your nose, then repeat the application.

Dosage for children aged 6 to 12 years: Spray once in each nostril, blow your nose and repeat the application.

After each use, clean the nozzle with a clean, damp cloth and dry carefully. It is recommended to apply Fitonasal Concentrated Spray several times a day, even at close intervals, repeating the application before going to bed.

WARNINGS: Do not use in case of hypersensitivity or allergy to one or more components. Use only nasally. Avoid contact with the eyes, and in case of contact, rinse with water. If symptoms persist, seek medical advice.

COMPOSITION: Standardised natural molecular complex composed of a tannic fraction and flavonoid fraction (from American witch-hazel and Thyme), as well as Rock salt (salt of mined extraction, not chemically treated), Water, Vegetable glycerine, Natural Eucalyptus flavour and Natural mint flavour.

EXPIRY: See the expiry date stated on the package. Do not use after the indicated expiry date. The expiry date indicated on the package remains the same even after the first application.

PACK FOR SALE: 30 ml glass bottle with nebuliser

Store at room temperature in a cool, dry place, away from heat sources and light. Keep out of the reach of children. The expiry date refers to the unopened product, stored correctly. Dispose of product responsibly after use.

Last revision date: 12/2019

FOR TODAY AND TOMORROW'S HEALTH

Aboca is a Benefit Corporation and certified B Corp.

We are committed to operating responsibly, sustainably and transparently towards the community and the environment. Impact report available at www.aboca.com

100% natural and biodegradable, Fitonasal Concentrated Spray does not contain synthetic, semi-synthetic or genetically modified substances.

PACKAGING MATERIALS		
Carton	PAP 21	> Paperboard
Leaflet	PAP 22	> Paper
Bottle with spray nozzle	C/GL 95	> Mainly Glass
Cap	LDPE 4	> Plastic
For disposal, please check the regulations in your local area.		

Aboca S.p.A. Società Agricola - Loc. Aboca, 20 - 52037 Sansepolcro (AR)
Italia - www.aboca.com

Company operating under certifications ISO 9001 and ISO 13485 for quality management system and ISO 14001 for environmental management system.

130002849/03