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Golamir

TREATMENT FOR SORE THROATS

Reduces pain and protects the mucosa

With **ActiFilmDOL**
Plant-based complex

MEDICAL DEVICE
Regulation (EU) 2017/745

NO-ALCOHOL SPRAY

ADULTS AND
CHILDREN
1 YEAR +

100%
NATURAL AND
BIODEGRADABLE
FORMULA

gluten
free

EN Golamir is a medical device that **acts on pain, soreness and difficulties swallowing** by **protecting the oropharyngeal mucosa** from the triggers of irritation. It physiologically **relieves pain and protects the throat** through an **indirect anti-inflammatory action** resulting from the following mechanisms of action:

Barrier

Antioxidant

- **Barrier**
Forms an active film on the mucosa to reduce hypersensitivity and protect it from attacks by external agents.
- **Antioxidant**
Reduces free radicals generated by inflammation of the mucosa.

Also creates an **environment unfavourable to the growth of bacteria**.

Pleasant forest fruit and blackberry flavour.

The composition and physiological mechanism of action of Golamir means that it soothes pain and protects the throat, while respecting the human body and the environment.

INDICATIONS

Treatment of irritation and inflammation of the oropharyngeal cavity in cases of sore throat, which may be caused by viruses, bacteria, allergens or contact with external irritants (smoke, smog, and dust). The product can also be used for colds when post-nasal drip may cause throat irritation. **From 1 year of age**. Can also be used in combination with medically prescribed pharmacological treatments.

INSTRUCTIONS FOR USE

Shake the bottle before use. It is recommended to administer (in the oropharynx):

1-6 years of age: 4 sprays up to 4 times a day.

Over 6 years of age and adults: 4 sprays up to 8 times a day.

In younger children, the product can be sprayed on the inside of the cheeks so that it is gradually absorbed. Inside the carton box there is a nozzle in a sealed sachet. On first use, remove the aluminium cap from the bottle and replace it with the nozzle.

Golamir can also be taken for prolonged periods. If symptoms persist, please consult your doctor or pharmacist. Do not use in case of individual hypersensitivity or allergy to one or more components. **Golamir** is generally well tolerated. However, if you notice any side-effects that may be attributable to the product or its use, please inform your doctor and/or pharmacist, as well as the Manufacturer Aboca, at the contact details provided in this leaflet. In the event of a serious incident, please also inform your country's competent authority. Based on the data available, there are no known or foreseen instances of overdose in the general population. Based on the data available, there are no foreseen interactions of **Golamir** with other medical devices, medicines or foodstuffs.

COMPOSITION

Golamir is a 100% natural and biodegradable product containing ActiFilm DOL, a plant-based complex. It does not contain any chemically synthesised, semi-synthesised or genetically modified substances and the production processes used to manufacture it do not employ solvents or any other chemically synthesised substances.

1 ml of product contains: **ActiFilm DOL** plant-based complex 25 mg, composed of **Tannins, Flavonoids, Resins and Polysaccharides** (LPME extractive fraction from Agrimony; Myrrh extractive fraction; Aloe vera dehydrated leaf gel); Vegetable glycerine; Water; Lemon juice; Xanthan gum; Apple juice; Natural forest fruit and blackberry flavours. Does not contain preservatives and artificial sweeteners. One spray corresponds to about 0.17 ml.

Aboca research has developed an exclusive technological extraction process, called LPME (Liquid-Phase Microextraction), which makes it possible to obtain complex, concentrated and validated extractive fractions from natural substances.

Golamir is evaluated against the following three parameters, by which Aboca assures the quality of these natural products in all of their complexity.

1. **Metabolomic digital fingerprinting using near-infrared spectroscopy (NIRS) technology**
confirming the natural product's production process validation
2. **Adhesion test**
confirming the barrier and protective action on the pharyngeal cells
3. **Tannin content $\geq 0.05\%$**

STORAGE AND EXPIRY DATE

Store at room temperature, away from heat sources and protected from light. Keep out of reach of children.

The expiry date refers to the intact product, stored correctly.

See the expiry date shown on the pack.

Do not use beyond the expiry date indicated.

It is recommended to keep the carton box.

PACK FOR SALE

30 ml glass bottle with spray nozzle.

LAST REVISION DATE: 11/2023

PACKAGING MATERIALS
For disposal, please check the regulations in your local area.

Carton box PAP 21 > Paperboard

Leaflet PAP 22 > Paper

Bottle GL 72 > Glass

Pump C/PP 92 > Predominantly Plastic

Bag PP 5 > Plastic

Cap C/ALU 90 > Predominantly aluminium

FOR TODAY AND TOMORROW'S HEALTH

For over 40 years, we at Aboca have been searching for the solutions to human health in nature, while at the same time respecting the human body and the environment.

ABOCA RESEARCH

We recognise the therapeutic potential of complex natural substances and bring them from the world of tradition to the world of innovation, according to Evidence-Based Medicine and Systems Medicine criteria.

100% NATURAL: FROM FIELD TO FINISHED PRODUCT

Our products are manufactured through a vertically integrated system that covers everything from organic farming and scientific research to production. Throughout this process, we exclude the use of chemically synthesised and non-biodegradable substances.

SUSTAINABLE HEALTHCARE

Health is not only for the good of the individual, it is for the good of all. That's why true health is not possible without sustainability. We are a Benefit Corporation, and through our work we are formally committed to bringing about benefit to the community and the environment. To find out more about all our products, visit www.aboca.com

Class IIa MEDICAL DEVICE 0477

Aboca S.p.A. Società Agricola
Località Aboca, 20 - 52037 Sansepolcro (AR) - Italia - www.aboca.com

GRLEA_GOMI-A