# Golamir

TRAJTIMI I DHIMBJES SË FYTIT

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# Golamir

TREATMENT FOR SORE THROATS

Reduces pain and protects the mucosa

With ActiFilmDOL  
Plant-based complex

MEDICAL DEVICE  
Regulation (EU) 2017/745

gluten free  
100% NATURAL AND BIODEGRADABLE FORMULA

## EXTRA STRENGTH SPRAY

**EN** Golamir is a medical device that **acts on pain, soreness and difficulty swallowing by protecting the oropharyngeal mucosa** from the triggers of irritation. It **physiologically relieves pain and protects the throat** through an **indirect anti-inflammatory action** resulting from the following mechanisms of action:

Barrier

Antioxidant

### • Barrier

Forms an active film on the mucosa to reduce hypersensitivity and protect it from attacks by external agents.

### • Antioxidant

Reduces free radicals generated by inflammation of the mucosa.

Its refreshing properties also reduce the feeling of discomfort in the throat.

It also creates an **environment unfavourable to the growth of bacteria.**

**The composition and physiological mechanism of action of Golamir mean that it soothes pain and protects the throat, while respecting the human body and the environment.**

## INDICATIONS

**Treatment of irritation and inflammation of the oropharyngeal cavity** in cases of sore throat, that can be caused by viruses, bacteria, allergens or contact with external irritants (smoke, smog and dust). The product can also be used for colds when post-nasal drip may cause throat irritation.

**From 6 years of age.** Can also be used in combination with medically prescribed pharmacological treatments.

## INSTRUCTIONS FOR USE

Shake the bottle before use. The recommended dose is 4 sprays into the oropharyngeal cavity 4 times a day. Inside the carton box there is a nozzle in a sealed sachet. On first use, remove the aluminium cap from the bottle and replace it with the nozzle.

**Golamir** can also be taken for prolonged periods. If symptoms persist, please consult your doctor or pharmacist.

Do not use in case of individual hypersensitivity or allergy to one or more components.

**Golamir** is generally well tolerated. However, if you notice any side effects that may be attributable to the product or its use, please inform your doctor and/or pharmacist, as well as the Manufacturer Aboca, at the contact details provided in this leaflet. In the event of a serious incident, please also inform your country's competent authority. Based on the data available, there are no known or foreseen instances of overdose in the general population. Based on the data available, there are no foreseen interactions of **Golamir** with other medical devices, medicines or foodstuffs.

## COMPOSITION

**Golamir** is a **100% natural** and **biodegradable** product containing ActiFilm DOL, a plant-based complex. No synthetic, semi-synthetic or genetically modified substances are used, and the manufacturing processes through which it is obtained also use no solvents or other chemically synthesised substances.

1 ml of product contains: **ActiFilm DOL** plant-based complex 390 mg, composed of **Tannins, Flavonoids, Resins** and **Polysaccharides** (LPME Agrimony extractive fraction; Myrrh extractive fraction; Aloe vera dehydrated leaf gel); Essential oils of Eucalyptus and Mint; Vegetable glycerine; Apple juice; Natural lemon and mint flavours; Xanthan gum; Water. Does not contain artificial preservatives and sweeteners.

One spray corresponds to about 0.17 ml.

**Aboca research has developed an exclusive technological extraction process, called LPME (Liquid-Phase Microextraction), which makes it possible to obtain complex, concentrated and validated extractive fractions from natural substances.**

**Golamir** is evaluated against the following three parameters, by which Aboca assures the quality of these natural products in all of their complexity.

- 1. Metabolomic digital fingerprinting using near-infrared spectroscopy (NIRS) technology**  
*confirming the natural product's production process validation*
- 2. Adhesion test**  
*confirming the barrier and protective action on the pharyngeal cells*
- 3. Tannin content  $\geq$  0.1%**

## STORAGE AND EXPIRY DATE

Store at room temperature, away from heat sources and light. Keep out of reach of children.

The expiry date refers to the intact product, stored correctly.

See the expiry date shown on the pack.

Do not use after the expiry date indicated.

It is recommended to keep the carton box.

## PACK FOR SALE

30 ml glass bottle with spray nozzle.

Alcohol 24.7% vol. - Fiscal Lic. IT00PGA00035A

**LAST REVISION DATE:** 11/2023

### PACKAGING MATERIALS

For disposal, please check the regulations in your local area.

**Carton box** PAP 21 > Paperboard

**Leaflet** PAP 22 > Paper

**Bottle** GL 72 > Glass

**Pump** C/PP 92 > Predominantly plastic

**Bag** PP 5 > Plastic

**Cap** C/ALU 90 > Predominantly aluminium

# Aboca

## FOR TODAY AND TOMORROW'S HEALTH

For over 40 years, we at Aboca have been searching for the solutions to human health in nature, while at the same time respecting the human body and the environment.

## ABOCA RESEARCH

We recognise the therapeutic potential of complex natural substances and bring them from the world of tradition to the world of innovation, according to Evidence-Based Medicine and Systems Medicine criteria.

## 100% NATURAL: FROM FIELD TO FINISHED PRODUCT

Our products are manufactured through a vertically integrated system that covers everything from organic farming and scientific research to production. Throughout this process, we exclude the use of chemically synthesised and non-biodegradable substances.

## SUSTAINABLE HEALTHCARE

Health is not only for the good of the individual. It is for the good of all. That's why true health is not possible without sustainability. We are a Benefit Corporation, and through our work we are formally committed to bringing about benefits to the community and the environment. To find out more about all our products, visit [www.aboca.com](http://www.aboca.com)

Class IIa MEDICAL DEVICE  0477

 **Aboca S.p.A. Società Agricola**  
Località Aboca, 20 - 52037 Sansepolcro (AR) - Italia - [www.aboca.com](http://www.aboca.com)

FSC

ALLEA\_GOMCA