## KONGJESTION NAZAL

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# Fitonasal Pediatric

## NASAL CONGESTION

### Rinses, decongests and protects the nasal mucosa

BALANCED AND STANDARDISED HYPERTONIC SOLUTION

No preservatives

MEDICAL DEVICE


Regulation (EU) 2017/745

**100%**  
NATURAL  
AND BIODEGRADABLE  
FORMULA

**6+**  
MONTHS

free

**600**  
mOsm/kg

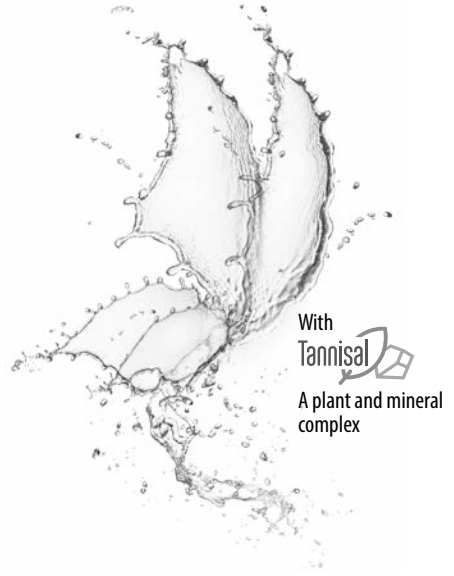With  
**Tannisal**  
A plant and mineral  
complex

#### NEBULISER SPRAY

**EN** Fitonasal Pediatric is a medical device that decongests, protects the nasal mucosa and clears the nose of excess mucus and the pathogens, irritants and allergens trapped within it. It performs a **physiological cleansing, decongestant and protective action** through the following mechanisms of action:

- **Balanced osmotic:** Helps draw back water from congested tissues without causing irritation and dryness, through its balanced and standardised osmolality (600 mOsm/Kg).
- **Local astringent:** It interacts specifically with certain inflammatory proteins found in the nasal fluid and enables their removal.
- **Barrier:** It forms a film on the mucosa and protects it from contact with irritants, helping to maintain the integrity of the epithelial barrier.
- **Antioxidant:** Reduces free radicals generated by inflammation of the mucosa.
- **Deep cleansing:** Helps remove excess mucus and the pathogens, allergens and other substances trapped within it thanks to the continuous-flow spray jet.

This is particularly helpful for infants, who are unable to blow their noses, and for children who do so inadequately.

Fitonasal Pediatric comes in a container featuring a spray dispenser with "**bag-on-valve**" (BOV) technology, which contributes to the effectiveness of the product and offers multiple benefits:

- **Easy to use** - continuous-flow spray in all positions;
- **Ergonomic** - spray nozzle ideal for even the most delicate nostrils;
- **Quality** - the product is contained inside a bag that is isolated from the external environment, which allows product out, but does not allow air, liquids or other residues back in;
- **Environmental protection** - a propellant gas (nitrogen) that respects the ozone layer.

**INDICATIONS:** Prevention and treatment of congestion and irritation of the nasal cavities caused by flu syndromes, colds, allergic rhinitis and both acute and chronic sinusitis. Using this product thus promotes breathing and aids nocturnal sleep in young children. Adjuvant treatment for coughs and sore throats caused by post-nasal drip and otitis media resulting from nasal congestion. From 6 months of age.

**INSTRUCTIONS FOR USE:** Use at least **twice a day**, morning and evening, and as often as deemed necessary.

#### From 6 months to one year of age

1. Lay your child down on one side, put the nozzle into the nostril that is now in the higher position and spray the product.
2. Turn your child onto their other side and repeat in the other nostril.

#### From 1 year of age

1. Put the nozzle into the nostril and apply gentle pressure for a few seconds.
2. Clean the nose or, if possible, blow it before spraying the product into the other nostril.

After each use, clean the nozzle with a clean, damp cloth and dry thoroughly. The product can be used frequently and for prolonged periods because it does not contain vasoconstricting decongestants, is non-addictive, does not irritate and does not dry out the mucosa. If **Fitonasal Pediatric** is used in combination with nasally applied medications, it is recommended that the two products be administered at least 1 hour apart. Do not take in cases of individual hypersensitivity or allergy to one or more components. Use nasally only. Avoid contact with the eyes, wash with water if necessary. If symptoms persist, please consult your doctor or pharmacist.

Fitonasal Pediatric is generally well tolerated. However, if you notice any side-effects that may be attributable to the product or its use, please inform your doctor and/or pharmacist, as well as the Aboca Manufacturer. In the event of a serious incident, please also notify the competent authority in your country.

Based on the data available, there are no known or foreseen instances of overdose in the general population.

Based on the data available, there are no foreseen interactions of **Fitonasal Pediatric** with other medical devices, medicines or foodstuffs.

**COMPOSITION:** Fitonasal Pediatric is a **100% natural** and **biodegradable** product containing Tannisal, a plant and mineral complex.

No synthetic, semi-synthetic or genetically modified substances are used, and the manufacturing processes through which it is obtained also use no solvents or other chemically synthesised substances.

1 ml of product contains: 6.8 mg of the natural **Tannisal** complex, containing **Tannins** (LPME extractive fraction of Witch Hazel - 0.2 mg) and **Minerals** (salt of mined extraction, not chemically treated - 6.6 mg); Water; Vegetable Glycerine; Lemon Juice; Nahcolite (natural Sodium Bicarbonate).

**The extractive fraction contained in the product is obtained through an exclusive technological process called LPME (Liquid-Phase Microextraction), developed by Aboca research.**

Fitonasal Pediatric is evaluated against the following parameters, by which Aboca assures the quality of these natural products in all of their complexity.

#### Metabolomic digital fingerprinting using near-infrared spectroscopy (NIRS) technology

confirming the natural product's production process validation

#### Osmolality 600 mOsm/Kg

confirming its decongestant efficacy and tolerability

**STORAGE AND EXPIRY DATE:** Store at room temperature, in a cool, dry place, away from heat sources and protected from light. The expiry date refers to the intact product, stored correctly. The expiry date indicated on the packaging remains unchanged even after the first spray. We recommend that you keep the carton box.

**Caution** - Pressurised container; may burst if heated. Do not pierce or burn even after use. Keep away from heat, hot surfaces, sparks, open flames or other sources of ignition. No smoking. Protect from sunlight. **Do not expose to temperatures above 50 °C/122 °F.** Keep out of reach of children.

**PACK FOR SALE** - Fitonasal Pediatric: 125 ml nebuliser spray

**LAST REVISION DATE:** 11/2024

# Aboca

#### FOR TODAY AND TOMORROW'S HEALTH

For over 40 years, we at Aboca have been searching for the solutions to human health in nature, while at the same time respecting the human body and the environment.

#### ABOCA RESEARCH

We recognise the therapeutic potential of complex natural substances and bring them from the world of tradition to the world of innovation, according to Evidence-Based Medicine and Systems Medicine criteria.

#### 100% NATURAL: FROM FIELD TO FINISHED PRODUCT

Our products are manufactured through a vertically integrated system that covers everything from organic farming and scientific research to production. Throughout this process, we exclude the use of chemically synthesised and non-biodegradable substances.

#### SUSTAINABLE HEALTHCARE

Health is not only for the good of the individual, it is for the good of all. That's why true health is not possible without sustainability. We are a Benefit Corporation. Through our work we are formally committed to bringing about benefit to the community and the environment. To find out more about all our products, visit [www.aboca.com](http://www.aboca.com)

#### PACKAGING MATERIALS

For disposal, please check the regulations in your local area

Carton box PAP 21 > Paperboard

Leaflet PAP 22 > Paper

Canister with nozzle C/ALU 90 > Predominantly Aluminium

Cap PP 5 > Plastic

Class IIa MEDICAL DEVICE

CE 0477